


# ACPi-11

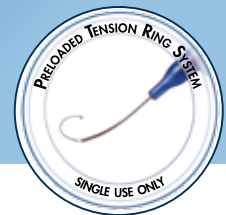

**ACPi-11** is a ready-to-use PMMA capsular tension ring in a preloaded single-use injector system. **ACPi-11** is used for repositioning of loose or desinserted zonulas in order to thwart the contraction strength of the capsular bag. Thus, it maintains the posterior capsule taut and capsular folds can be avoided.

## Advantages:

- Pre-loaded
- Single use
- Ready to use in the operating theatre
- Time-saving system

## Indication:

- Cataract surgeries of subluxated lenses
- Zonular desinsertion
- Zonular weakness
- Risk of capsular retraction
- High myopia
- Prevention of capsular bag shrinkage in patients with congenital cataract

## Specification:

- Diameter: 11 mm
- Material: PMMA
- One piece
- Sterilization: ETO

## Instructions for use:

1. Press firmly on the plunger of the injector to unlock it (see picture 1), then carefully release the pressure on the plunger to allow the ring to completely enter the cannula (see pictures 2 and 3).
2. Introduce the cannula into the capsular bag.
3. Gently press on the plunger to inject and coil the ring in the bag (see pictures 4 and 5).
4. Liberate the ring by releasing the suture from the fixation hole.
5. Allow the fixation suture to enter the cannula before removing the injector from the eye.

## Contact:



PREMED PHARMA KFT.  
CÍM 2040 Budaörs, Gyár u. 2.  
TELEFON 06 23 889 700  
FAX 06 23 889 710  
E-MAIL [info@premedpharma.hu](mailto:info@premedpharma.hu)  
WEB [www.premedpharma.hu](http://www.premedpharma.hu)

